

# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

---


This Quick Guide provides a summary on speech tests available on Senti and Sentiero devices. This includes the following test procedures:

- Speech test platform with recorded speech, e.g. for determination of speech recognition threshold (Universal Speech Test module, Recorded Speech (CD Player) module)
  - o English (US): Spondees Child/Adult, Northwestern University Auditory Test No. 6 (NU-6), Central Institute for the Deaf Wordlist No. 22 (CID W-22), Phonetically Balanced Kindergarten Word Score (PBK-50), Maryland Consonant-Nucleus-Consonant (Maryland CNC)
  - o Dutch: NVA test
  - o French: Mots français CAD
  - o German: Freiburger
  - o Hungarian: Beszéd Teszt
  - o Slovenian: Govorni
  - o Spanish: Dr. Tato
- Live Speech
- Speech Understanding in Noise (SUN)
- Mainzer Audiometric Test for Children (MATCH)
- Munich Auditory Screening Test for Processing Disorders (MAUS)
- Bochum Auditory Speech Discrimination Test (BASD)

In the following the basic device workflow and the different speech tests are described shortly. For more information on each speech test, please refer to the How-To-Manual on the PATH MEDICAL website (<https://pathme.de/support/>).

### **SPEECH AUDIOMETRY TESTS WITH SENTI / SENTIERO DEVICES**


In order to start a speech test connect an appropriate transducer (e.g. headphones, insert earphones, or free-field loudspeakers) to the blue device socket.

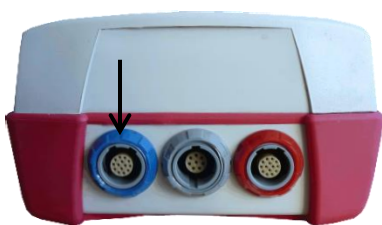

# QUICK START GUIDE


## SPEECH AUDIOMETRY TESTS

Select "Speech" from the start menu. Depending on your device license, you will have several speech tests available:

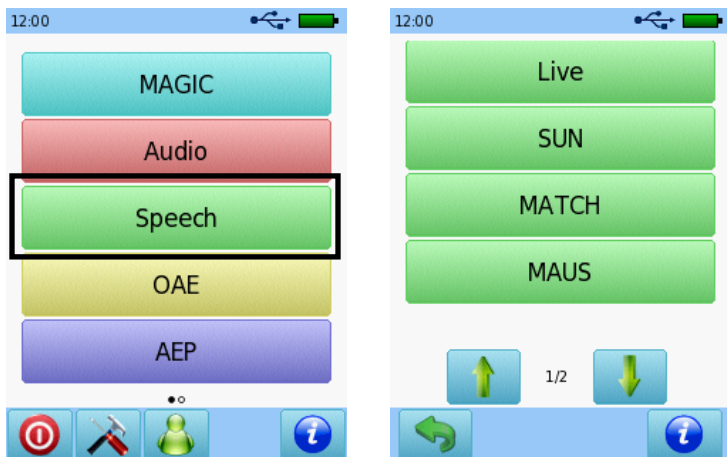


**Universal Speech Test (UST):** Provides a platform for determination of speech recognition thresholds with recorded word lists in several languages (e.g. Spondees Child/Adult, NU-6, CID W-22).


Select the respective speech test (please note that on the speech test selection screen the test name is shown, not UST). When using e.g. the NU-6, CID W-22, PBK-50, or Maryland CNC lists, the level selection is available within the main test screen so that the level can be changed while going through a single word list. The test result is presented as a table and graph and can be switched by pressing the result screen.

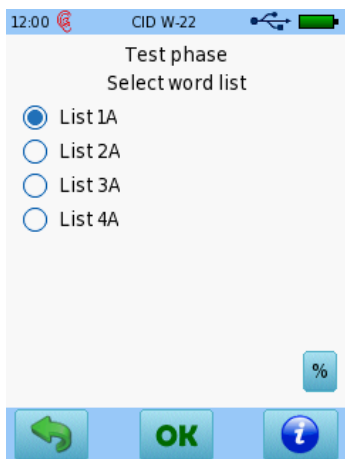

Word list selection

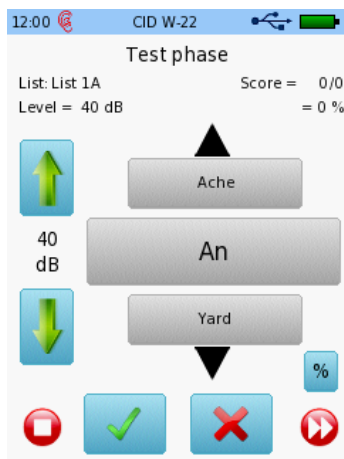

Level selection and scoring

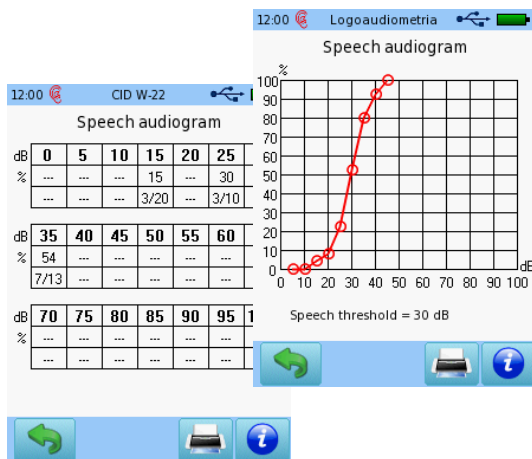


Result views (table, graph)


# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS


When using e.g. the Spondees Child or Spondees Adult lists, select a word list, and change the speech level until a speech threshold is obtained. The determined speech threshold is the level which is set by the examiner when finishing the test with the stop button.

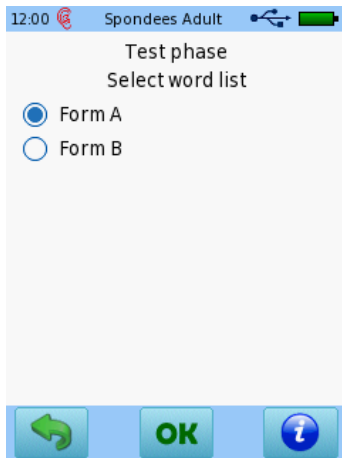

*Word list selection*

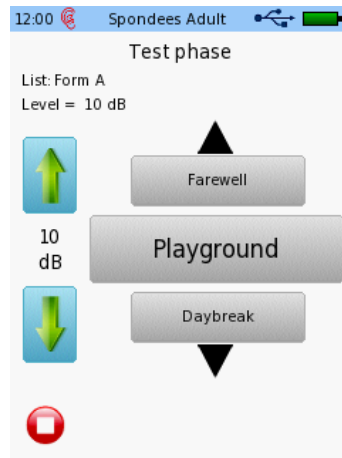


*Threshold determination*

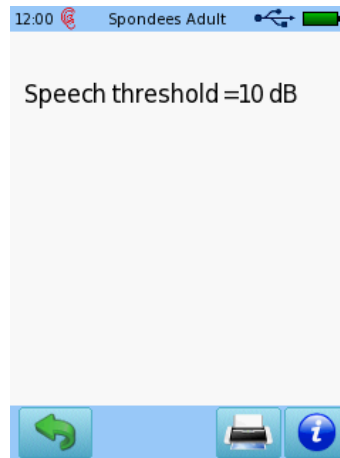


*Result view*

**Recorded Speech (CD Player):** Provides the ability to select a CD track and play the words from this track with start, stop, and pause functions.


Select "Recorded", configure stimulus and noise, and select the sub-test and track you want to use. After the track selection, the test starts automatically by playing the words from the chosen track as on a CD player. The tested subject has to repeat the word, and the examiner then evaluates and enters the response.

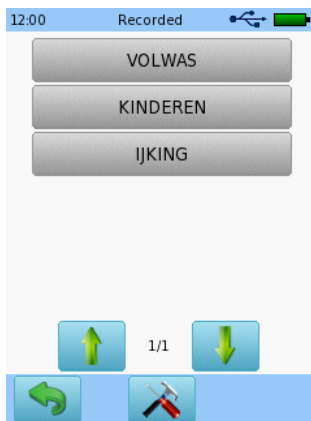

*Sub-test selection*

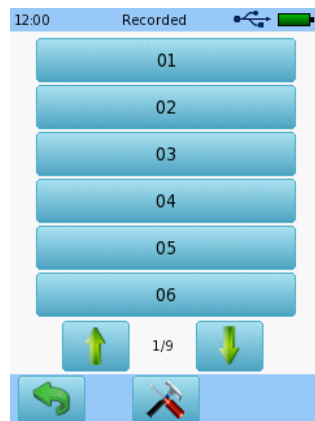


*Track selection*

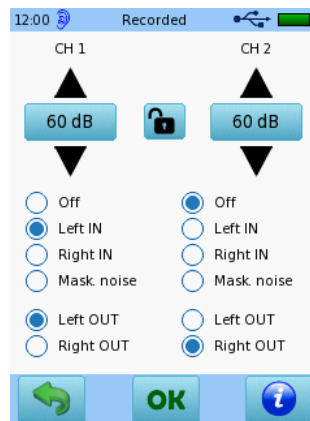

*Stimulus/noise configuration*

# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

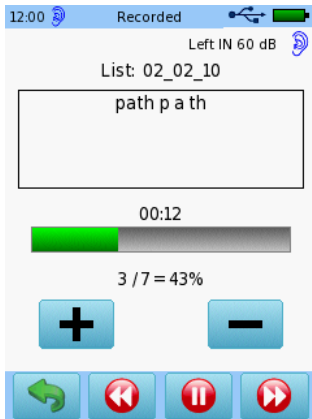

*Test user interface (replay of words within track, scoring)*

When leaving the test, the result screen is shown as a graphical or tabular speech audiogram similar to the UST result view (see above).

**Live Speech:** Provides a platform for determination of speech detection threshold, speech recognition threshold, and word recognition with live speech presented by the examiner via microphone.

Select "Live" and select the test mode (SDT: Speech Detection Threshold, SRT: Speech Recognition Threshold, WR: Word Recognition) by pressing the test mode button. Adjust the microphone input level if necessary. Once the settings are correct, start the test by speaking into the microphone at the desired level. Use the appropriate buttons to set the stimulus level, accept the threshold, or adapt the score.



*SDT user interface*



*SRT user interface*



*WR user interface with accepted threshold*


# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

The result screen shows a graphical or tabular speech audiogram similar to the UST result view (see above) including the word recognition score and the determined speech detection and recognition threshold.

**Speech Understanding in Noise (SUN):** Provides a self-controlled speech in noise test aiming at the identification of consonants by presenting vowel-consonant-vowel (VCV) logatoms. Currently available pronunciations are: US English, Italian, and German.

Select "SUN" and configure the test parameters (e.g. fixed/adaptive procedure) in the test module settings. After starting the test, different logatom alternatives are presented and the tested subject is asked to select the button, which corresponds to the word that has been understood. The result is shown as score (fixed procedure) or threshold (adaptive procedure).

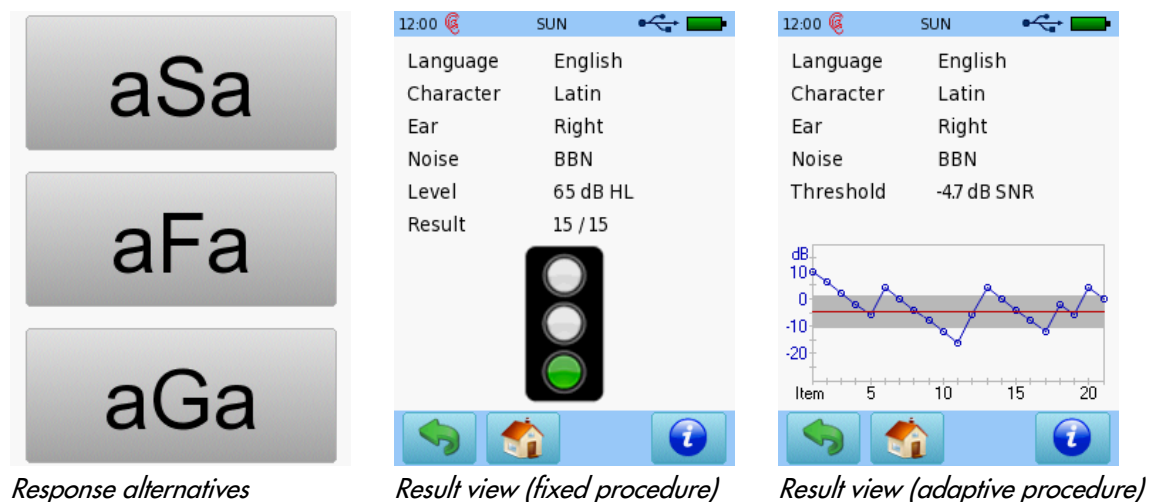

**Mainzer Audiometric Test for Children (MATCH):** Provides an image-based self-controlled speech test aiming at the identification of words representing the typical vocabulary of a two-year-old child. The test can be performed in quiet or in noise. The test is currently available in German.

Select "MATCH" and configure the test parameters (e.g. fixed/adaptive procedure, with/without noise) in the test module settings. After starting the test, a box is presented, which needs to be pressed for starting the stimulus. Four response alternatives are presented as images. The tested subject is asked to select the image, which corresponds to the word that has been understood. The result is shown as score (fixed procedure) or threshold (adaptive procedure).

# QUICK START GUIDE


## SPEECH AUDIOMETRY TESTS




Start stimulus



Response alternatives

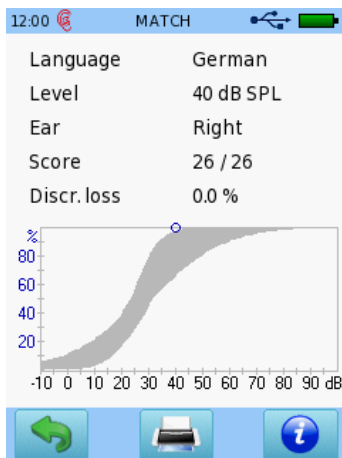

Result view (fixed procedure)

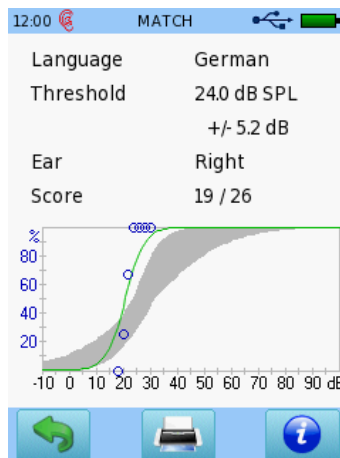

Result views (adaptive procedure)




**Munich Auditory Screening Test for Processing Disorders (MAUS):** Provides a screening test for auditory processing disorder. The test consists of three sub-tests: sequence of syllables, words in noise, and phoneme differentiation and identification. The test is currently available in German.


Select "MAUS" and follow the instructions on the screen to perform the three different sub-tests. The result is presented as score for each sub-test.

# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

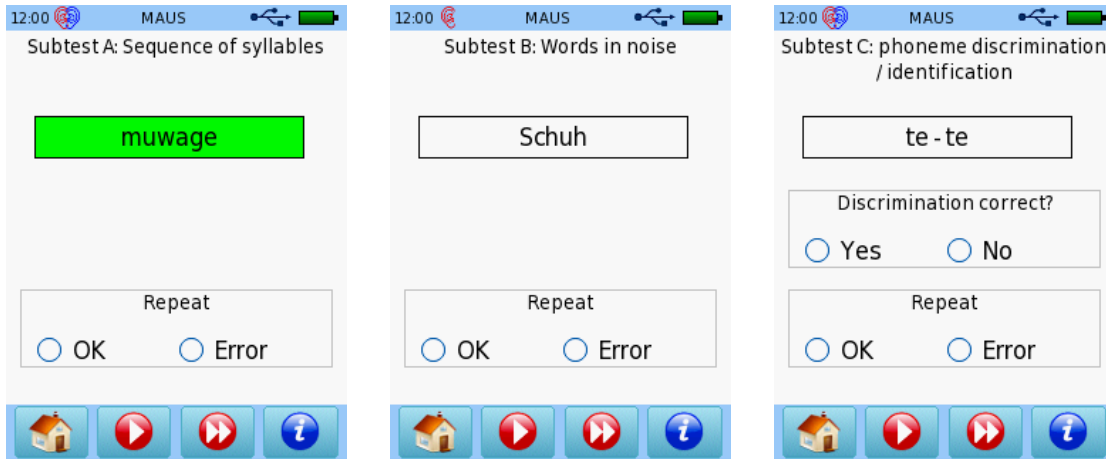

*Different MAUS sub-test user interfaces*

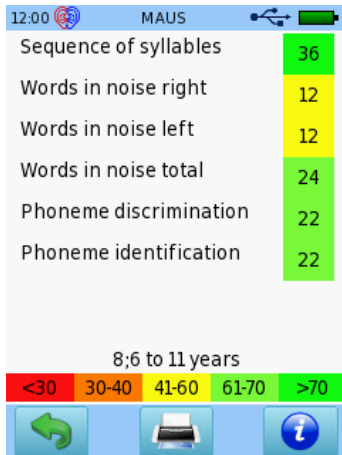


*Result view (scores)*

**Bochum Auditory Speech Discrimination Test (BASD):** Provides a test battery for determination of speech development deficits and related auditory processing disorders. The test includes multiple sub-tests (differentiation of consonants, just noticeable differences in frequency, level, duration, and amplitude modulation). The “differentiation of consonants” sub-test is currently available with German pronunciation.


Select “BASD” and configure the appropriate discrimination task in the test module settings. Three stimuli are presented. Two of these three stimuli are identical, i.e. they feature the same consonant, level, frequency, duration or amplitude modulation. The two identical stimuli shall be indicated by the tested subject by uncovering the corresponding two cards.

# QUICK START GUIDE


## SPEECH AUDIOMETRY TESTS

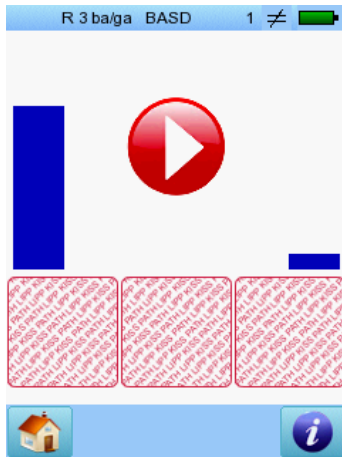

*BASD user interface*

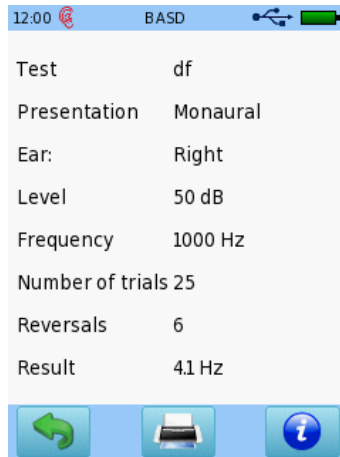

*Result view (discrimination)*

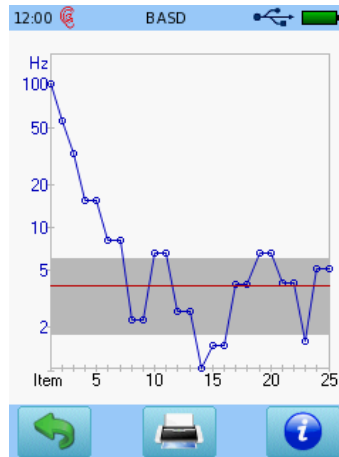

*Result view (time course)*

### **SPEECH AUDIOMETRY WITH FREE-FIELD LOUDSPEAKERS**

Speech audiometry tests (except Live Speech, MAUS, BASD) can be performed using free-field loudspeakers. In order to use the free-field loudspeakers, connect the cable to the blue connector on the device and to the loudspeakers.



*Free-field loudspeaker connection cable*


From the main screen, press the Tools  button to access the "Settings" menu.


Select "Functional Checks" and then "Free Field Calibration". The menu will guide you through the calibration process.



# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

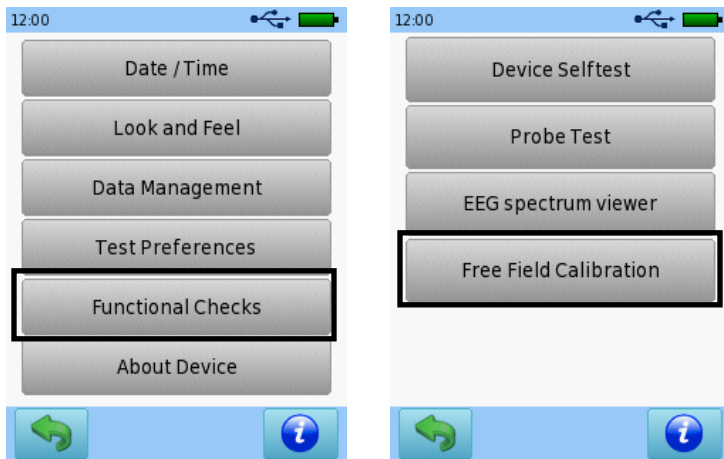

A detailed explanation on the free-field loudspeaker calibration procedure is available on the device when pressing the  button.

### **SPEECH TEST LICENSE OVERVIEW**

SNAM	SUN (US English)	UT01	Logaudiometria Dr. Tato (Spanish)
SNIT	SUN (Italian)	UT02	Freiburger (German)
SNDE	SUN (German)	UT06	Govorni (Slovenian)
MHDE	MATCH (German)	UT08	Spondee Child (US English)
MSDE	MAUS (German)	UT09	Beszéd Teszt (Hungarian)
LPDE	BASD (German)	UT10	Spondee Adults (US English)
		UT11	NU-6 (US English)
		UT12	CID W-22 (US English)
		UT13	PBK-50 (US English)
		UT14	Mots français CAD (French)
		UT15	Maryland CNC (US English)

If you are interested in further speech test implementations please contact PATH MEDICAL ([info@pathme.de](mailto:info@pathme.de)).

# QUICK START GUIDE

## SPEECH AUDIOMETRY TESTS

---

### Quick Guide Information

Release date: 2018-02  
Revision: 01  
Valid from: Firmware Rev. 2.3, Mira PC Software Rev. 1.8.0  
Applies to: All Senti / Sentiero devices

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH MEDICAL reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH MEDICAL for further information.

The latest revision of the Quick Start Guide is available online at [www.pathme.de/download](http://www.pathme.de/download).

Errors and omissions excepted.

### Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH MEDICAL GmbH.

Copyright © 2018 PATH MEDICAL GmbH

PATH MEDICAL GmbH  
Landsberger Straße 65  
82110 Germering  
Germany

Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: [www.pathme.de](http://www.pathme.de)

