

QUICK START GUIDE SENTIERO HANDHELD DIAGNOSTIC (BLUE LABEL)

TURNING ON/OFF, CHARGING THE DEVICE




On/off
button on
right side



Power
supply
connector

START MENU

Note: this screen (test options) can vary depending on your set up

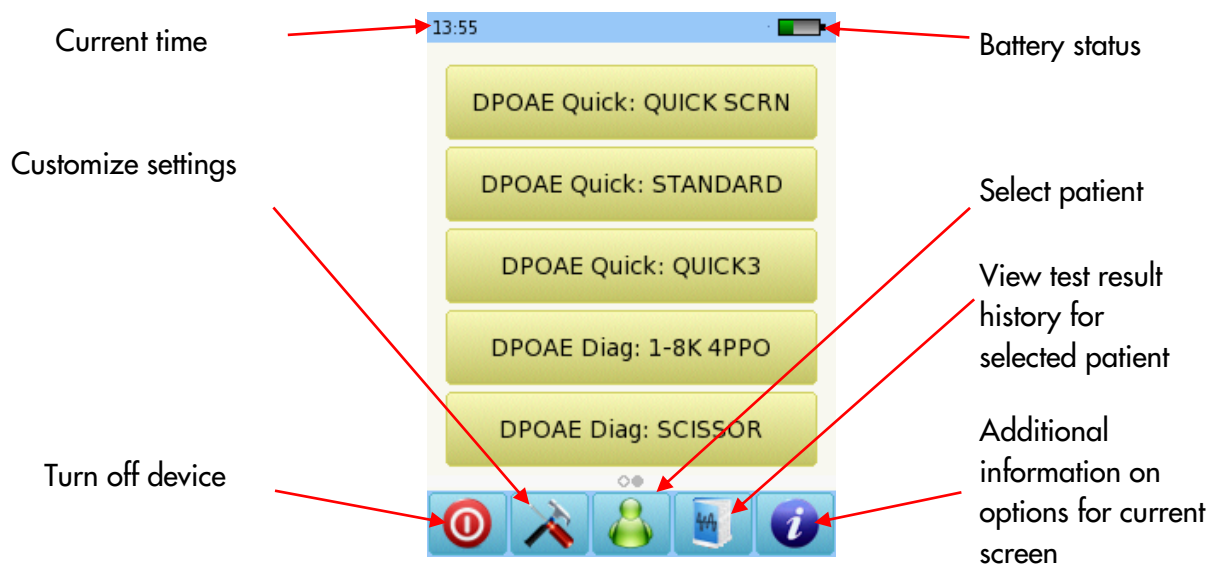

TESTING

DISTORTION PRODUCT OTOACUSTIC EMISSIONS (DPOAE) SCREENING


Connect the probe to the red connector. Select one of the Presets for DPOAE Screening.




QUICK START GUIDE SENTIERO HANDHELD DIAGNOSTIC (BLUE LABEL)

Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select the test ear by pressing either the Right or Left button. Cartoon mode will start the test showing a cartoon instead of the progress bar.

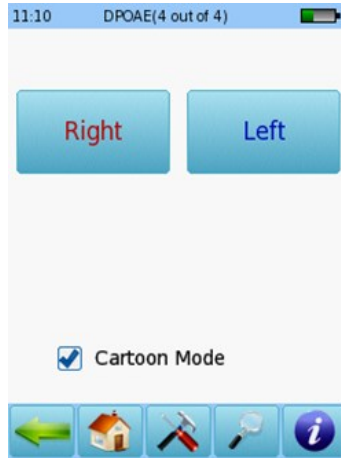


A calibration will be performed prior to measurement. A green light indicates successful calibration (below left). After successful ear probe calibration, the test will start automatically.

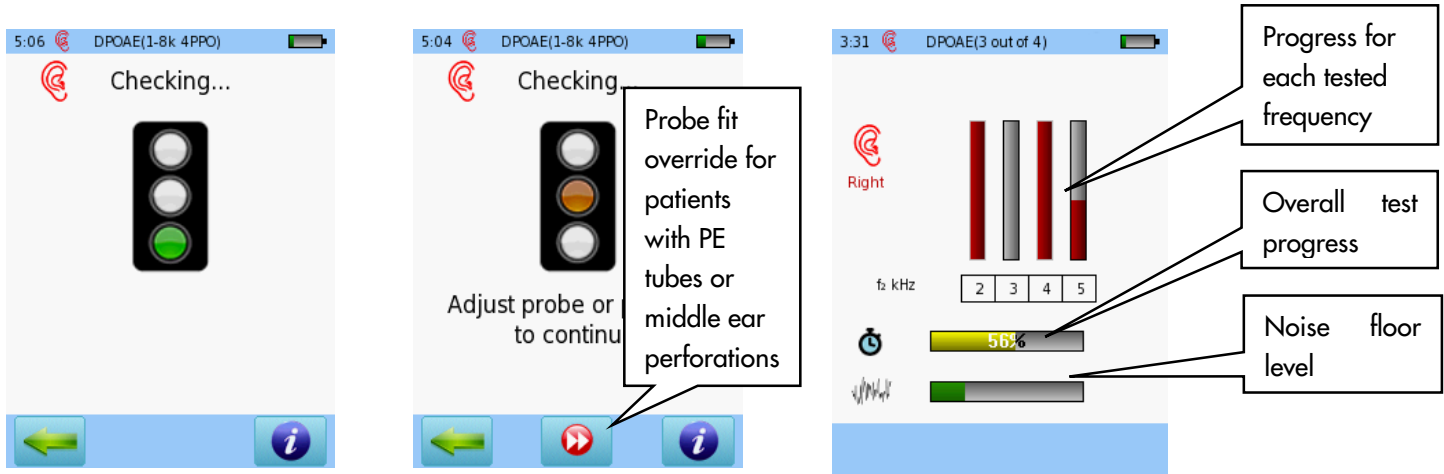

QUICK START GUIDE SENTIERO HANDHELD DIAGNOSTIC (BLUE LABEL)

DPOAE SCREENING TEST RESULTS

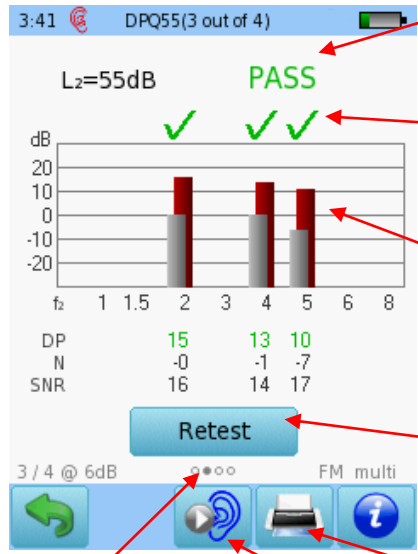

Overall screening test result


Green check marks indicate valid response. Red circles indicate no valid response

Red (right ear) or blue (left ear) bars show response amplitude. Grey bars show noise amplitude

Retest frequencies that did not pass

Print results to a label or to a pdf file

Test other ear

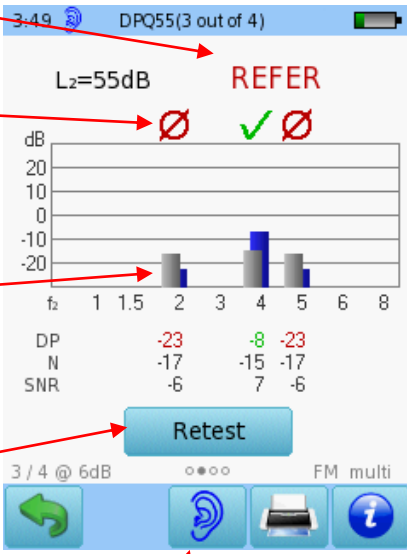

Overall screening test result

Green check marks indicate valid response. Red circles indicate no valid response

Red (right ear) or blue (left ear) bars show response amplitude. Grey bars show noise amplitude

Retest frequencies that did not pass

Print results to a label or to a pdf file

Switch to results of the other ear after both ears have been tested

Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views.

DPOAE DIAGNOSTIC / HIGH RESOLUTION


Connect the probe to the red connector. Select one of the Presets for DPOAE Diagnostic.




Select the appropriate ear tip size and place it on the probe tip. Insert the probe in the ear making sure that the ear tip fits tightly into the ear canal.

Select the test ear by pressing either the Right or Left button. Cartoon mode will start the test showing a cartoon instead of the progress bar, this can be changed during the test by tapping on the plane icon at the bottom of the device screen.

QUICK START GUIDE SENTIERO HANDHELD DIAGNOSTIC (BLUE LABEL)

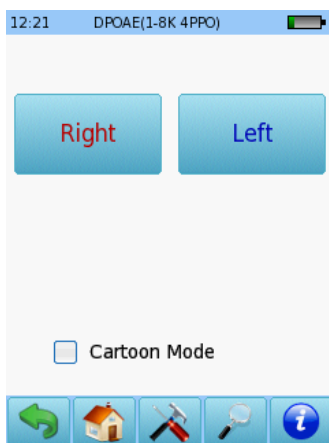


A calibration will be performed prior to measurement. A green light indicates successful calibration. After successful ear probe calibration, the test will start automatically. Please note that the presets for DPOAE diagnostics are HiRes, which allows testing frequencies in 10 Hz increments or up to 30 points per octave.

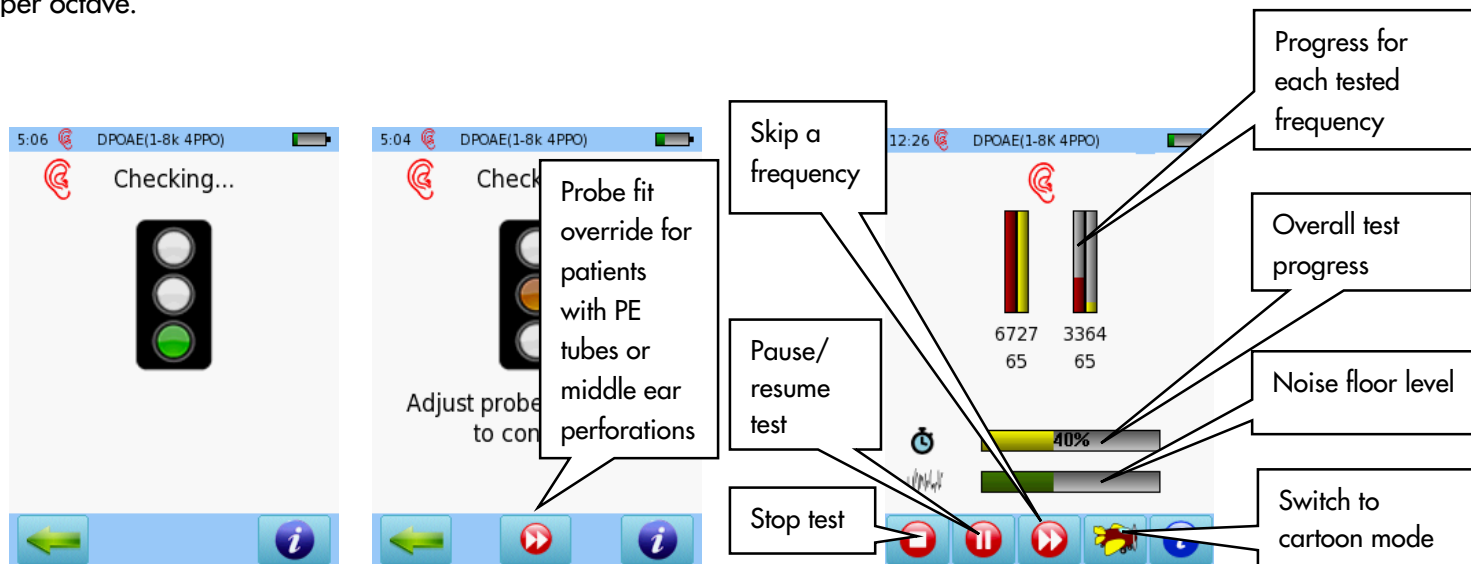

QUICK START GUIDE SENTIERO HANDHELD DIAGNOSTIC (BLUE LABEL)

DPOAE DIAGNOSTIC TEST RESULTS

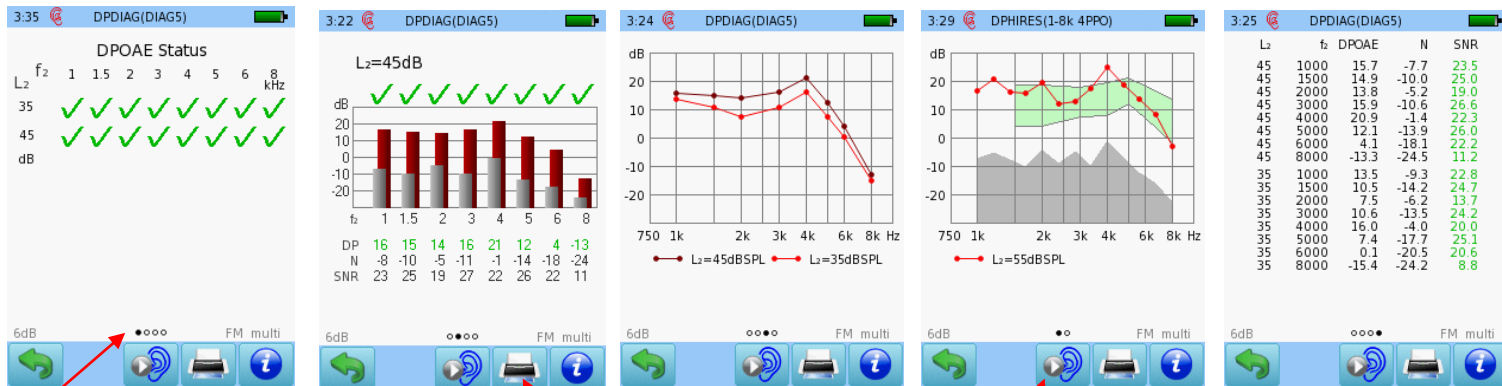

Note: These circles indicate that other screens are available to view. Swipe your finger right or left to get other views.

Print results to a label or to a pdf file

Test other ear

Green check marks indicate valid response

Red or blue bars indicate response, grey bars indicate noise

Outcome for each test sorted by f2 and L2. Multilevel test results shown. Tap on screen to view details for each level tested.

High resolution test results (4 PPO)

Numeric values shown in table format

QUICK START GUIDE

SENTIERO HANDHELD DIAGNOSTIC

(BLUE LABEL)

Quick Guide Information

Release date: 2017-01-11
Revision: 02
Valid from: Firmware Rev. 2.1, Mira PC Software Rev. 1.6.4
Applies to: Model: SOH100098, web description SOH02

All mentioned items, products, brands and trademarks are registered or owned by the mentioned companies.

All information, illustrations, and specifications provided within this manual are based on the latest product information available at the time of publication. PATH medical reserves the right to make changes at any time without notice.

Please note that not all products displayed on our webpage or advertising materials are available in all territories. Please contact your local distributor or PATH medical for further information.

The latest revision of the Quick Start Guide is available online at www.pathme.de/download.

Errors and omissions excepted.

Copyright Notice

No part of this manual may be reproduced, translated, stored, or transmitted, in any form or by any means, electronic, mechanical, photocopying, recording, or otherwise, without the prior written consent of PATH medical GmbH.

Copyright © 2016 PATH medical GmbH

PATH medical GmbH
Landsberger Straße 65
82110 Germering
Germany

Tel.: +49 89 800 76 502 Fax: +49 89 800 76 503 Internet: www.pathme.de

CE
0124